

NEPTUNE

For hemostasis following needle/sheath removal

DIRECTIONS FOR USE:

Manual Pressure – **NEPTUNE PAD:** Part # 8870-01

- Place 1-2 drops of sterile ringers, saline or a balanced salt solution on NEPTUNE.
DO NOT SATURATE.
- Follow facility protocol for manual pressure. Place **NEPTUNE PAD** over skin puncture site and apply pressure toward arteriotomy.
- Upon hemostasis, cover NEPTUNE with absorptive dressing.
- NEPTUNE may remain in place up to 24 hours. To remove NEPTUNE, saturate with saline and remove slowly.
- Dispose of used NEPTUNE per facility policy.

Mechanical Pressure – **NEPTUNE DISC:** Part # 8850-01 or 8860-02

- Place 1-2 drops of sterile ringers, saline or a balanced salt solution on NEPTUNE.
DO NOT SATURATE.
- Follow facility protocol for using mechanical devices (all "C" clamps and NEPTUNE EZ Hold). Place **NEPTUNE DISC** over arteriotomy with NEPTUNE over skin puncture site.
- Upon hemostasis, moisten NEPTUNE on sides and through disc access holes, remove disc leaving NEPTUNE and cover with absorptive dressing.
- NEPTUNE may remain in place up to 24 hours. To remove NEPTUNE, saturate with saline and remove slowly.
- Dispose of used NEPTUNE per facility policy.

CONTRAINDICATIONS:

NEPTUNE is not intended for use:

As a surgical implant, surgical hemostat, nor in related, full-thickness trauma involving muscle, tendon or bone;

On third degree burns;

On ulcerating lesions due to mycotic or venereal infections.

CAUTIONS:

If irritation occurs on the puncture site after application of NEPTUNE, discontinue use and see a doctor before further use.

Single Use Only.

TZ Medical Inc.
7272 SW Durham Rd #800
Portland, OR 97224
(800) 944-0187