



TZ MEDICAL SOFTGELS

GENERAL CARE & USE

Overview:

The TZ Medical SoftGels provide superior cushioning and positioning, ensuring optimal exposure to the surgical site.



SoftGels Heating and Cooling

- Consult your facility protocol for recommended heating and cooling ranges identified for patient safety.
- Use extreme caution when heating products to temperatures above normal body temperature of 98.6°F
- Blanket warmers and forced air warming devices (i.e. Bair Hugger) are recommended for heating.
- Refrigerators or freezers are recommended for cooling.

SoftGels Cleaning

- SoftGels should be cleaned and disinfected after every use.
- General hospital approved disinfectants that do not contain alcohol may be used
- Do not use any disinfectants containing alcohol or other undiluted substances. This may lead to staining or hardening of the gel product.

