

## COMFORT SHIELD CLEANING INSTRUCTIONS

The TZ Medical **Comfort Shield** is made of strong, durable vinyl material that will withstand many uses; however, care must be taken when cleaning the product to keep it in good condition.

### Washing:

1. **Prewash** to remove any caked-on material by rinsing with water and wiping with a non-abrasive cloth or sponge.
2. **Wash** the Comfort Shield with a mild detergent (such as ENZOL<sup>®</sup> Fast-acting Enzymatic Cleaner) by briefly spraying and wiping down. Isopropyl alcohol (rubbing alcohol) may be used to spot clean only but **NOT** for soaking. Avoid abrasives such as brushes or scouring pads as they may scratch the surface.
3. **Rinse** off all cleaning agents. *Only clean the Comfort Shield with chemicals which are indicated for cleaning vinyl surfaces.*
4. **Dry** using a cloth, air, or heat up to 120 °F.

### CAUTION:

- **DO NOT allow the Comfort Shield to be heated above 120 °F.** If a Comfort Shield is suspected to have been heated above 120 °F, inspect for visible damage (melted, deformed, or discolored areas).
- **DO NOT steam clean** or allow the Comfort Shield to come into contact with steam as it can degrade the materials of the product.
- **DO NOT clean the Comfort Shield with any chemical not specifically indicated for the cleaning of vinyl surfaces**
- **DO NOT use a Comfort Shield that has evidence of damage to any part of the product.**