

# TZ MEDICAL IR PAD CLEANING INSTRUCTIONS

Approved products and methods for cleaning and disinfecting of TZ Medical IR Pads.

**Do not apply tape to any pad.**

## Cleaning soils and stains:

Wash with a generous application of neutral soap suds and lukewarm water.  
Rinse with water and dry. Do not immerse.

-or-

Use a diluted bleach solution of 10-parts water to 1-part bleach. Wipe dry with a clean cloth. **Do not immerse. Bleach MUST be diluted to the 10:1 ratio.**

All chemicals/solutions MUST be diluted to the manufacturers recommendations.  
**No chemical or solution should be left to dry on the pad surface.** After the allotted time has passed to insure disinfection, the pad surface should be wiped again with a wet cloth to remove the remaining chemical or solution.

## Disinfecting:

Use Lysol Brand III I.C. Disinfectant Aerosol Spray by Reckitt Benckiser Inc.,  
Montvale, NJ -- per manufacturer's instructions.  
Reference: Reckitt Benckiser UPC # 36241-95029-3

-or-

Use a diluted bleach solution of 10-parts water to 1-part bleach. Wipe dry with a clean cloth. Do not immerse. Bleach MUST be diluted to the 10:1 ratio.

All chemicals/solutions MUST be diluted to the manufacturers recommendations.  
**No chemical or solution should be left to dry on the pad surface.** After the allotted time has passed to insure disinfection, the pad surface should be wiped again with a wet cloth to remove the remaining chemical or solution.